

# Hy spray

## User manual



Guidance for a proper use  
of the Hyspray

# User manual

The disinfection with the Hyspray and the HyPro Technical is a complement to manual cleaning/disinfection. The disinfection by air is done without human or animal presence.

## Recommendations:

- Close the windows and open the closets/wardrobe.
- Put the Hyspray on a stable prop or trolley.
- Take off the head of the spraying pipe by pressing the inserting hook and screw the flask containing the product (1 litre) into it.
- For any manipulation of products, always wear protective gloves and protective glasses.
- Put back the head of the spraying.
- Plug in the appliance; connect the power cable in the socket provided to that effect in the appliance and in the electrical outlet.
- Direct the spraying head of the Hyspray towards the area to be treated.
- Press the button On/Off . The display indicates an initial value for a room of a volume of 50 m<sup>3</sup>.
- Set the volume of the room to be treated by selecting the button UP or DOWN. A short pressure on these buttons make vary the volume by 5 m<sup>3</sup>. An extended pressure leads to a more important variation. The capacity of adaptation of the volume of the room to be treated varies by 5 m<sup>3</sup> to 165 m<sup>3</sup>
- Press the button to set in motion, a sound signal triggers for 90 seconds, which gives sufficient time to leave the room that is about to be disinfected. The display screens the countdown from 90 seconds to 0 second.
- At the end of the countdown, the Hyspray starts the disinfection process.
- Wait for the spraying to be done (the average time of the spraying is 11 minutes per 50 m<sup>3</sup>) and observe the recommended time for the H<sub>2</sub>O<sub>2</sub> to disperse, which lasts at least 60 minutes. The residual H<sub>2</sub>O<sub>2</sub> concentration after the disinfection process varies according to the room temperature, the volume of the room and the air inlet. In case of doubt, it is recommended to use an appliance of measurement to determine the H<sub>2</sub>O<sub>2</sub>.
- Press the button ON/OFF to switch the appliance off.
- Take off the flask of the HyPro technical during any haulage.

## Reminder:

- Consumption: 6,6 ml/m<sup>3</sup> set
- For the curative disinfection, set the volume of the room.
- Contact time after spraying (until complete disinfection): 1 hour
- Wait to enter the room until the spraying is over.

## Troubleshooting:

If the display shows an "ERROR", it certainly means that the device has been unplugged improperly. In such context, push on the start button for 10 seconds duration and the display will revert to the original screen showing 50 m<sup>3</sup>.